# femometer Vinca II

### MENU

1 Who Should Use Femometer

2 How to Use Thermometer

3 Alerts & Sounds

4 About the Display

6 Product Maintenance & Care

5 Troubleshooting

7 Technical Specifications

8 Product Functions & Features 9 Safety State ment

10 Disclaimer

11 Copyright Statement

12 Warrenty

13 Symbol Key

14 Record of Purchase

# Pair Femometer with Your Smartphone

Step 1: Remove the cap from



Step 2: Turn on your phone's Bluetooth.



Step 3: Follow the prompts on



Step 3: Close the tray from the

längere Zeit nicht benutzen.

### 4 Replace the Battery

Step 1: Insert the stainless steel tool into the crevice at the base of the thermometer, gently pry open the battery tray



Step 2: Wait for one minute, put a CR2032 battery in the battery tray with "+" facing down.

base of the thermometer.

• Entfernen Sie den Akku, wenn Sie ihn



# 4. About the Display

ove the upper cover of the thermometer. ## will be shown on the screen. nent will start when the LED backlight is on and you hear a beep.

2. To extend the battery life, the screen will be off once "JFF" is displayed 2 seconds after the measurement.

	Symptoms	Possible reasons	Solutions
	Lo	While measuring, the temperature is lower than 32.00°C (90.08°F).	If the device keeps showing <code>Lo</code> during measurement, please put the cover back, wait for one minute and measure again.
	H,	While measuring, the temperature is higher than 42.00℃ (107.60°F)	
	Err	Hardware failure	Remove the battery and wait for one minute. Put the battery back. If the message comes up again, please contact the retailer for solutions.
	Ó	Lit up and flashing	Please change the battery (CR2032).

# 5. TROUBLESHOOTING

Symptoms	Possible reasons	Solutions
I never heard the end	Thermometer is off.	Close the cover and reopen it to restart the thermometer.
tone while the thermometer is under my	Measurement time is too short.	Keep measuring for 1–3 minutes .
tongue.	Button battery has run out.	Replace the button battery.
The data did not sync to the app after my	The Bluetooth is not open.	Open the Bluetooth on the phone.
measurement.	The phone is too far to sync.	Put the phone close to the thermometer.
There is no sound when opening the thermometer.	Button battery has run out or the thermometer is broken.	Replace the button battery and if the thermometer still makes no sound, the thermometer is broken.

### 7. TECHINCAL SPECIFICATIONS

Vinca II FM-VC-201

120.2mm\*24.4mm\*16.4mm Weight 24.8g (without battery) Transmission distance Up to 16 feet

Measurement 89.6°F-107.6°F Accuracy ±0.09°F(95.0°F-100.4°F) ±0.18°F(89.6°F-95°F) & (100.4°F -107.6°F) CR2032

Humidity: ≤93%

Storage conditions Temperature: -40-131°F Apple devices: iPhone 5 or above with version at least iOS 8.0, iPhone 4/4s excluded. Android devices: 4.3 or above and supports Bluetooth 4.0/4.1 3-6 months

Operating conditions Temperature: 41°F-104°F Humiditv: ≤85% Atmospheric pressure: 70Kpa–106Kpa 11 EN

# 10. DISCLAIMER

- This manual is written in accordance with available information and contents are subject to change without notice. Bangtang has made the best effort to guarantee the manual's accuracy and reliability. However, the company does not assume liability for loss or damage caused by omissions.

inaccuracies or typographical

errors in this manual.

- If there are discrepancies in this manual due to software upgrades, instructions and function dictated by the software should prevail. If improvements are made to the product or the product technology changes, please visit Bangtang's technical support website for further information.
- All symbols and logos ciated with Bluetooth. including the Bluetooth word Bluetooth SIG. Any use of such marks by Bangtang is under license. Other trademarks and
  - combination mark and Bluetooth Smart and Bluetooth ready marks, are all trademarks owned by the

trade names used belong to

their respective owners.

# 11. Copyright

- Hangzhou Bangtang Network Technology Co., Ltd. (Bangtang) Behält sich alle Rechte vor und hat das finale Recht inne, dieses Handbuch und die darin enthaltenen Statements zu interpretieren und zu interpretieren und zu überarbeiten. Das Copyright dieses Handbuchs liegt bei Hangzhou Bangtang Network Technology Co., Ltd. (Bangtang) Ohne weitere Zustimmung darf kein Unternehmen und kein Individuum dieses Handbuch ven vervielfältigen, kopieren ode Informationen übersetzen.
- Technology Co.,Ltd. (Bangtang) Floor 12,92 Yueda Street, Binjiang District, Hangzhou, China Manufacturer/Hersteller:
- Joytech Healthcare Co., Ltd. No.365, Wuzhou Road, Yuhang Economic Development Zone, Hangzhou city, 311100 Zhejiang, China
- Hangzhou Bangtang Network

# - Dieses Produkt unterliegt einer Garantie für ein Jahr für Werksfehler ab Datum des Kaufs. Dies deckt keine Schäden ab, die durch Unfälle, falsche Nutzung, Entfremdung, kommerzielle Nutzung oder unautorisierte Anpassungen und Reparaturen des Produkts verursacht werden. Bitte richten Sie sich für den Umtausch an die Stelle bei der Sie das Produkt gekauft haben und behalten Sie die Originalrechnung, um Ihren Kauf zu beweisen. Für unseren Kundensupport, besuchen Sie bitte unsere Firmenwebseite unter www.bongmi.com.

2. HOW TO USE FEMOMETER

12. Garantie

# 28 DE

### 1. WHO SHOULD USE FEMOMETER

This product should be used by women to measure basal body temperature. Patient is the intended operator

1 Dowmload App Search "Femometer" on Apple Store or Google play to

Android 4.3+ Bluetooth 4.0/4.1



Google Play

Step 3: Open the application, the

data will be transmitted

1 EN

3 EN

5 EN

EN

9 EN

13 EN

### **3** Take Your Temperature

Step 1: Set a reminder on your phone to wake up and take your temperature each morning.

thermometer from its case, place it

under your tongue and hold it there for 1-3 minutes, until you hear 3 beeps, indicating temperature has

Step 2: Upon waking, remove the





### Upload Date to the App

Step 1: Once your temperature has been taken, your phone's Bluetooth must be on, the data can be transmitted from the thermometer to your phone.

Step 2: The data from the thermometer will sync automatically to the Femometer App. If the data does not sync, remove the cap from the thermometer, and tap the scale ring in the Femometer App to

# 3. ALERES & SOUNDS

- When you remove the thermometer from the case, a single beep indicates that it is
- When you close the case, a single beep indicates that the thermometer is off.
- When taking your temperature, 3 consecutive beeps indicate that the measurement is complete
- A beep lasting longer than 2 seconds indicates an error Please close the case and reopen the thermometer until

the measurement is complete.

Symptoms	Mode	Description
fo,	Fast Mode	Duration: approximately 30 seconds Advantage: fastest Suitable for: users with good temperature measuring habits
Fo.	Normal Mode	Duration: 1 - 1.5 minutes Advantage: optimal balance between stability and speed Suitable for: Femometer newbles
Lo•	Mercury Mode	Duration: 3 minutes Advantage: safest Suitable for: users who want to follow NFP rule exactly to avoid pregnancy

### 6. PRODUCT MAINTENANCE & CARE

### 1. Battery care

- Replace the battery when instructed via notificath Femometer App.

- Always dispose of used

- 2. Cleaning & Disinfection
- Clean the tip of the thermometer with a solution of 75% rubbing alcohol and 25% water before and after every use
- batteries according to local laws and regulations. - Do not use cleaning agents
  - other than alcohol and water to clean the device as it may damage or decrease the lifetime of the product and/or present safety risks.

### 8. PRODUCT FUNCTIONS & FEATURES 9. SAFETY STATEMENT

### 1. Basal body temperature measurement & Bluetooth

-The thermometer measures basla body temperature to 1% Your basal body temperature is automatically uploaded to your phone via Bluetooth, which is fast and energy-efficient, and represented via temperature Charts and graphs to track and

12 EN

- 2. Data stored in the cloud - All your data is stored in the cloud, so you don't have to
- worry about manually uploading or losing your information. You can also sync the thermometer 3. Share date with friends - Share your data easily with make predictions about your your partner or doctor menstrualcycle. directly from the Femometer

- Never submerge the device in

water - this can damage the device and result in erroneous

- 1. The EMC of the product is up To standard of IEC60601-1-2.
- 2. The material that contacts Skin is nontoxic, nonirritating And up to standard of IS010993-1,-5,-10.

 Hangzhou Bangtang Network Technology Co.,Ltd (Bangtang). reserves ail rights, and retails the final right to inerpret and revise this handbook and the statements contained therein. Copyright of this manual belongs to Hangzhou Bangtagn belongs to Hangzhou Bangtagn Network Technology Co., Ltd (Bangtang). Without prior written permission, no company or individual is allowed to reproduce, copy or translate this

many or any information

ontained therein.

EN

11. COPYRIGHT STATERMENT

- Distributor: Biosmart Teknoloji A.Ş., Eti Mah. Ali Suavi Cad. 9/A Çankaya, Ankara / TÜRKİYE
- Hangzhou Bangtang Network Technology Co.,Ltd Floor 12, 92 Yueda Street, Binjiang District, Hangzhou. China

App via Twitter or Facebook.

# 12. WARRANTY

This product is warranted from manufacturing defects for we year from the date of retail purchase. It does not cover damages or wear resulting from accident, misuse, abuse, commercial use or unauthorized adjustment and repair of product. Please direct all returns to the place of original purchase and retain your original receipt, as you may be asked to provide a copy fix proof of purchase. To find the customer service menu, please visit our company website at www.mothersmart.com.tr

- This product is warranted from

# 13. SYMBOL KEY

# ★ Device type is BF Model Prchased:

Read user manual before use Manufacturer

SN Serial number

Serial Number: Phone Number: Data of Purchase:

14. PURCHASING RECORD

No littering Recyclables E-Mail Address:

15 EN